


Spenco RX®

ORTHOTICARCH

Orthotic Arch Supports

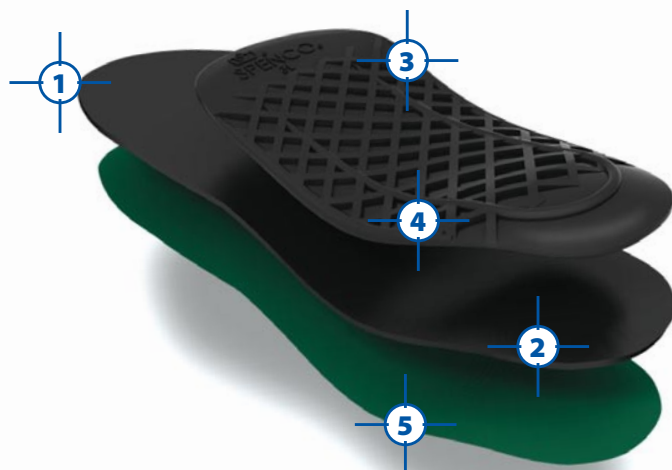

Full Cat. No. 43-042
3/4 Cat. No. 43-158



Firm, adjustable arch support.

Target Consumer:

Consumers with flat, low, or fallen arches.
Consumers seeking firm arch support to control foot position.

Recommended For:

Treatment of arch pain. May be molded for a custom fit to support high arches.
To help reduce pronation and supination for improved stability.
Choose $\frac{3}{4}$ length for additional toe room in tighter fitting shoes.



Best For Use In:

Casual and athletic shoes (or boots).



Components*:

Nylon, SpenCore® Material, Fabric, Arch Height: 25.80mm (1 in), Thickness at Heel: 6.29mm (1/4in)



Instructions:

Place in shoes, green side up. Trim with scissors if necessary. Adjust the shape of the orthotic by placing in boiling water for 2 minutes and then bending to desired shape.

Caution: Do not handle heated orthotic without protecting your hands. Use gloves or a towel to re-shape. Allow orthotic to cool before placing in shoe.



Care:

Hand wash in warm water. Air dry.

Sizing Chart:

Size	0	1	2	3	4	5	6
U.S. Women's	3/4	5/6	7/8	9/10	11/12		
U.S. Men's			6/7	8/9	10/11	12/13	14/15
U.K.	2/3	4/5	5/6	7/8	9/10	11/12	13/14
European	32/34	36/38	38/40	40/42	42/44	44/46	46/48

Features & Benefits:

- 1 Full Length** provides complete heel-to-toe comfort.
- 2 SpenCore® Material** Exclusive cushioning layer absorbs shock and impact to provide maximum comfort.
- 3 Moldable Arch Insert** The flexible plastic arch insert helps support and control the foot.
- 4 7 Sizes** for correct arch placement for better overall fit, comfort and performance.
- 5 4-way Stretch Fabric with Silpure Antimicrobial** helps prevent blisters while controlling odor.

One-Year Unconditional Guarantee

Distributed by:

NELSON JAMESON

INC.

800-826-8302 nelsonjameson.com

*Measurements are approximate and vary by size.

Silpure is a Registered Trademark of Thomson Research Associates, Inc.