PortaReader™

QUICK-START MANUAL AND SETUP GUIDE*

Doc. 695 Rev. 2022-001

Clear, Quantitative Results for
Subclinical Mastitis Screening

For use with PortaCheck PortaSCC®
45-minute somatic cell count test kits.

*To get the most out of your new
PortaReader™, download the full instruction
manual at portacheck.com/portareader/ifu,
or follow the QR code.

INTRODUCTION

Congratulations, and thank you for choosing the PortaReader™ from PortaCheck! You are now the owner of an advanced digital tool designed to simplify your work and increase the production and milk quality of your dairy herd.

The PortaReader™ is a convenient handheld unit that provides quantitative results when used with PortaCheck PortaSCC® SCC 45-minute test kits (sold separately). With these results, users can quickly and easily detect subclinical mastitis in fresh bovine milk samples right on the farm. The PortaReader™ features a rugged construction with a bright color that's easy to locate in the field.

The PortaReader™ is manufactured in the U.S. by DCN Dx for PortaCheck. It is covered under a one-year standard warranty.

INCLUDED

- 1 PortaCheck PortaReader™
- 1 Quick-Start User Guide
- 2 AAA batteries

REQUIRED

- PortaCheck PortaSCC® 45-minute somatic cell count test kits (product number 1K096 and 1K212) (not included)

OPTIONAL


- MicroUSB cable (current rating above 1.5A)
- 5V power adapter

INTENDED USE

The PortaReader™ is a handheld testing instrument that is used exclusively with PortaCheck PortaSCC® 45-minute somatic cell count test kits (not included, sold separately). When used with the PortaSCC test strips, the PortaReader™ provides a quantitative readout of the estimated somatic cell count in a milk sample. This test system is intended to be used solely with fresh bovine milk. The system is not a laboratory reference method and should not be used as a diagnostic test. Consult a veterinarian before starting any treatment.

The PortaReader™ is intended to be used on-farm or in a laboratory and to be stored in indoor environments. The reader is not intended to be left outdoors for extended periods of time nor to be used with any other test strips beyond those described in this document. The device is not intended for human use. The device is not intended for use with any other PortaSCC® test kits, including the PortaCheck PortaSCC® Quick Test.

PORTAREADER™ COMPONENTS

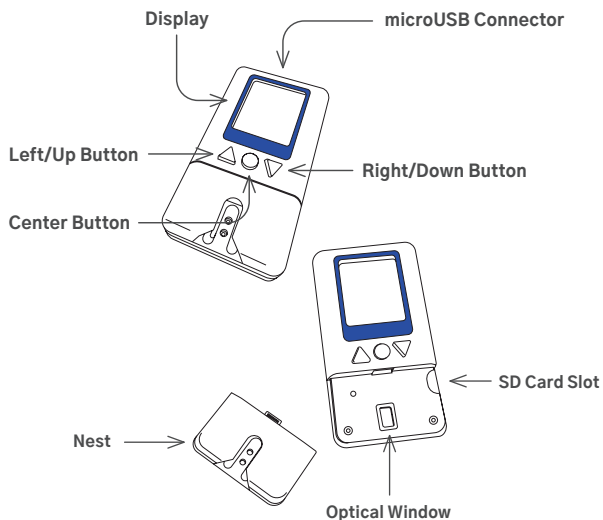PRIOR TO USE

The PortaReader™ should be inspected prior to use for signs of damage. Operating the PortaReader™ with any of the following characteristics may result in unsafe conditions or inaccurate results.

- Cracks in the housing
- Missing or damaged screen cover
- Missing or damaged battery cover
- Missing or damaged test strip nest
- Scuffed or scratched screen or screen cover
- Damage to power adapter port or microUSB cable (not included)

Do not attempt to replace or service any components on the PortaReader™. If a component is missing or broken, contact PortaCheck for servicing.

PORTAREADER™ SETUP

Powering the PortaReader™

Before using the PortaReader™ for the first time, install two AAA batteries (included). The battery compartment is located on the back of the PortaReader™ unit. Before replacing the battery cover, ensure the batteries are installed with the correct polarity (i.e., positive to positive, negative to negative).

On subsequent uses, the PortaReader™ can be powered with two AAA batteries, a 5V power adapter and microUSB cable (not included), or both.

Expired batteries should only be replaced with alkaline AAA cells.

Set the Time/Date and Other Functions

For instructions on setting the time and date and other functions, download the full PortaReader™ instruction manual at portacheck.com/portareader/ifu, or follow the QR code below.

Display Navigation

The control panel of the PortaReader™ features two directional arrows and a center button. To navigate the display, use the “up” and “down” arrows to highlight options. Press the center button to select the option. To return to the previous screen, use the “back menu” option.

RUNNING TESTS

IMPORTANT

The PortaReader™ is designed to be used only with PortaCheck PortaSCC® 45-minute somatic cell count test kits. Ensure you have read the test strip instructions, which can be found at

portacheck.com/product/portascc-milk-test

or follow the QR code below.

IMPORTANT

In order to provide the most accurate results, ensure that the optical area is clean and clear of debris prior to taking a reading. We recommend cleaning the optical area after every 30 test strips or before each use.

To read tests on the PortaReader™:

1. Press the center button to power the PortaReader™ (Fig. 1).

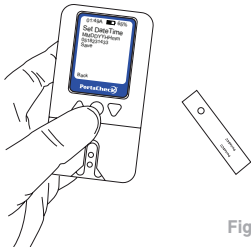
Fig. 1

CAUTION

Before inserting test strips in the PortaReader™, ensure you have read and understand any instructions that came with the test strips. Check the optical area for lint or debris and clean it if necessary.

2. When “RUN” appears on the display screen, insert the PortaSCC® test strip. The sample well must face down (Fig. 2).

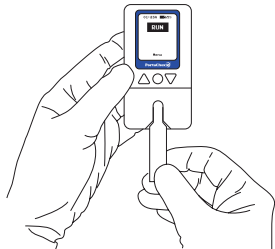
Fig. 2

3. Press the center button to measure the test strip (Fig. 3).

Fig. 3

4. Read the value displayed on the screen (Fig. 4).

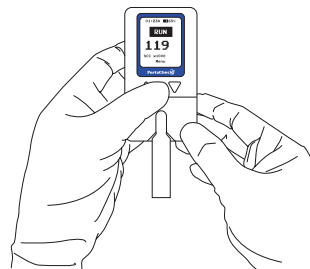
Fig. 4

INTERPRETING THE RESULTS

The PortaReader™ detects SCCs between 50,000 and 1,500,000. Results are displayed in the following configuration:

<####> SCC x1000

To determine the SCC results of your test, multiply the number (shown in the example as ####) by 1,000. (For example, a result of “180 SCC x1000” means the sample has an SCC of 180,000 cells/mL.) If the sample has an SCC above or below the acceptable range, the PortaReader™ will display a result of “LOW” or “HIGH.”

PortaReader™ Test Results

LOW	< 50,000 cells/mL
Numerical Value	50,000 – 1,500,000 cells/mL
HIGH	> 1,500,000 cells/mL

RECOMMENDED OPERATING CONDITIONS

For best performance, the PortaReader™ should be operated in the following conditions:

Temperature	15°C–35°C
Relative Humidity	20–80%
Use Environment	On the farm or indoors, keep out of direct sunlight

STORAGE CONDITIONS

For best results, store the PortaReader™ in the provided packaging.

Temperature	5°C–45°C
Relative Humidity	10%–85% (with no condensation)

SAFETY AND REGULATORY COMPLIANCE

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. Changes or modifications not expressly approved by PortaCheck could void the user’s authority to operate the equipment.

The instructions and safety information in this user manual must be followed to ensure safe operation of the PortaReader. Please note that if the equipment is used in a manner not specified by DCN Dx, LLC, the protection provided by the equipment may be impaired.

DCN Dx, LLC, is not responsible for outcomes due to misuse of the reader resulting in damages to personnel or property, which will be the responsibility of the end user. Use of the equipment in a manner not specified by the manufacturer may impair the protection provided by the equipment.

PROPER USE

CAUTION

- Do not move the PortaReader™ while a test is in progress.
- Do not immerse the PortaReader™ or its components in any liquid.
- Wipe spills off the PortaReader™ before inserting the test strip in the nest.
- Do not drop, hit, or place objects on top of the PortaReader™ or its components.
- Do not open or tamper with PortaReader™ components. The PortaReader™ may only be serviced by authorized personnel.
- Do not unplug the SD card while the PortaReader™ is on.
- Do not remove any screws on the case. Doing so will void the warranty.

ELECTRICAL SAFETY

The PortaReader™ was designed and manufactured to comply with the international regulations listed in this section and left the factory in a safe condition.

The instrument complies with the protection requirements of IEC 61326-1:2005. It is the user's responsibility to ensure that a compatible electromagnetic environment for this instrument is provided and maintained in order that the reader will perform as intended.

Do not use this reader near sources of strong electromagnetic radiation (e.g., unshielded intentional RF sources) as these may interfere with the proper operations.

Unscrewing the housing of the instrument may expose voltage-carrying components.

WARNING

- Only use UL-listed AC/DC USB power supplies to power the PortaReader™.
- Do not attempt to maintain, repair, or open the PortaReader™ while it is plugged in.
- Do not open the battery door while the PortaReader™ is plugged in.
- If you suspect that the PortaReader™ can no longer be operated safely, turn it off and take steps to ensure that no one will subsequently attempt to use it.

CARRYING AND HANDLING

- For best results, only operate the PortaReader™ while in a stationary position. Refrain from or limit movement while taking a reading.
- If using a microUSB power cable, ensure the cable is unplugged from the PortaReader™ prior to moving the unit. Failure to do so may result in damage to the PortaReader™, the power cable, or other components.
- Store the PortaReader™ in an upright position.

CLEANING

WARNING

Turn off and unplug the reader prior to cleaning.

1. To clean the PortaReader™ housing, use a water dampened microfiber cloth and wipe across the surface of the reader until clean.
2. To clean the PortaReader™ optic area, first remove the nest (Fig. 10). To remove the nest, gently apply pressure and pull the nest down and away from the display. Do not lift the nest up and away from meter as this could damage the nest. To clean the optic area, use a microfiber cloth dampened with water. Make sure the area is completely dry before replacing the nest.

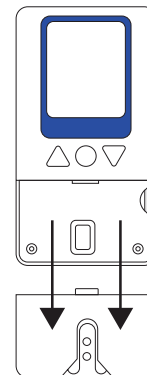
Fig. 10

Removal
of the
Nest

3. Use the dry parts of the microfiber cloth to dry any remaining liquid on the surface prior to attempting to plug in or turn on the reader.

CAUTION

Ensure no liquid enters the nest port.

CAUTION

Do not use paper towels to wipe down the reader's display. These may cause scratches on the display cover.

CAUTION

Do not directly apply liquids to the reader or use large amounts of liquid to wash.

TROUBLESHOOTING AND ERROR CODES

Error Message	Description	Troubleshooting
Battery Low	The batteries are nearing the end of their life. The user can still scroll through the menu and view results but will not be able to take a reading.	Replace the batteries.
Shutting Down	The batteries have expired. The RTC will continue to operate. The unit will remind the user to replace the batteries.	Replace the batteries.
Read Error	The PortaReader™ has misread the strip.	Reinsert the strip and take another read. If error continues, restart reader.
Unkn Error	An unidentified error has occurred	Reinsert the strip and take another read. If error continues, restart reader.

Exp Date	Unit has not been QC'd properly or has experienced a substantial hardware or software malfunction	Restart reader.
QR ERROR!	Software malfunction	Restart reader.

If any of these errors persist after troubleshooting, or if an error message displays that is not listed here, contact PortaCheck for support.

WARRANTY

The PortaReader™ is protected under a standard one-year warranty.

The warranty is voided if the PortaReader™ suffers damage or malfunction due to abuse, misuse, neglect, negligence, accident, improper storage, use contrary to the provided documentation and specifications, or unauthorized alterations.

Procedure for Warranty Coverage

To be eligible for replacement under this warranty, the Purchaser must (I) promptly contact the Seller to report the non-conformance, (II) cooperate with the Seller in confirming or diagnosing the non-conformance, and (III) return the Product to Seller (transportation charges prepaid) according to the Seller's instructions.

For warranty claims, contact PortaCheck.

CUSTOMER SERVICE

For PortaReader™ questions, troubleshooting, support, or warranty claims, contact:

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